BC SEALER ION⁺

ENGLISH

DEFINITION

Bioceramic root canal sealer ready for use.

INDICATIONS

1. Root canal obturation of permanent teeth.
2. Treatment of internal resorption.

INTENDED USE

Ready-to-use bioceramic endodontic cement.

INTENDED USER

BC SEALER ION⁺ is intended for use by trained dentists to perform endodontic procedures.

INTENDED ENVIRONMENT

Dental offices.

INTENDED POPULATION

Patients with permanent teeth with closed apex, diagnosed with pulpal and periapical disease.

CONTRAINDICATION

- Do not use in patients with known sensitivity to the formula components.
- Do not use the product after expiration date.
- Do not use if packaging is damaged.

COMPOSITION

Calcium and Magnesium Silicate, Calcium Sulfate, Potassium Sulfate, Calcium Oxide, Zirconium Oxide, Silicon Dioxide, and dispersing agent.

TECHNICAL INFORMATION

- Ready-to-use biocompatible product.
- Setting time: 30 – 240 minutes.
- High radiopacity.
- High alkalinity.
- Can be used with conventional techniques, single cone, and thermoplastic obturation.

WARNINGS

- Use rubber dam for all indications.
- Do not use BC SEALER ION⁺ without gutta percha cones.
- The product is moisture-sensitive, therefore close packaging adequately with proper pressure to avoid drying.
- Dry the canal with paper points without over-drying. Moisture from dentin tubules initiates material setting.
- Use the product only after controlling infection. Acidic pH from infected medium may impair material setting in canal.
- Avoid excessive product extrusion into periapical region.
- Disinfect applicator tip prior to use. Discard applicator tip after use.
- Do not reuse applicator tips. Clean and disinfect syringe for subsequent uses to prevent cross-contamination.

NOTE: Any serious incident related to the device must be reported to the manufacturer and relevant health authorities where the user and/or patient is established.

PRECAUTIONS

- Use protective eyewear, mask, and gloves when handling the product. In case of contact with eyes or soft tissues, rinse with plenty of water.
- Immediately after the use of BC SEALER ION⁺ return syringe properly closed in the packaging (pouch), along with the silica gel pack. The package must be correctly sealed and stored according to "STORAGE" section.
- For subsequent use, check if the silica gel pack inside the envelope remains blue.
- Safety and efficacy of use have not been established in pregnant

- women, lactating women, children, or adolescents.
- Keep out of reach of children.

TECHNIQUE OF USE

USE OF APPLICATOR TIPS

1. It is recommended to use a disposable plastic cover on the syringe to avoid cross-contamination. Disinfect the applicator tip prior to use.
2. Confirm material flow from syringe before direct application into the canal.
3. Position the applicator tip and syringe assembly properly into the canal and press plunger gently to avoid excess product extrusion.
4. Before closing syringe, withdraw plunger to stop flow.
5. Clean excess material from syringe tip and close with adequate pressure after use to prevent moisture entry that causes product drying.
6. Clean and disinfect syringe for subsequent uses to prevent cross-contamination; in case of contamination with saliva or blood, discard it.

A. OBTURATION OF ROOT CANALS OF PERMANENT TEETH

TRADITIONAL TECHNIQUE - LATERAL COMPACTION

1. Anesthetize, place rubber dam, open the crown and then perform the biomechanical preparation of the canal.
2. Do the final irrigation of the canal with sodium hypochlorite and EDTA.
3. Select the most suitable digital spacer.
4. Dry the canal only with absorbent paper points without causing excess dryness.
5. Remove sealing cap and attach the applicator tip to the syringe
6. Apply BC SEALER ION⁺ with gradual retraction of the syringe to achieve complete canal filling.
- ① *IMPORTANT: Press the plunger gently to avoid excess product extrusion.*
7. Introduce gutta percha cone according to the last used file.
8. With main cone adjusted, adjust digital spacer to create space for insertion of accessory cones loaded with obturating cement.
9. Repeat this process until digital spacer no longer penetrates.
10. Radiograph to verify proper canal filling.
11. Cut the cone to desired height with heated instruments followed by vertical compaction.
12. Remove excess material from canal walls with water, perform coronal sealing and restoration.

- ① *IMPORTANT:*
- BC SEALER ION⁺ *does not contract upon setting.*
- BC SEALER ION⁺ *can be used with conventional techniques, single cone.*
- *Use conventional gutta removal techniques.*

SINGLE CONE TECHNIQUE

1. Anesthetize, place rubber dam, and then perform biomechanical preparation of the canal.
2. Remove the sealing cap and attach the applicator tip to the syringe.
3. Dry canals with absorbent paper points without completely removing moisture.
4. Apply BC SEALER ION⁺ with gradual retraction of the syringe to achieve complete canal filling.
- ① *IMPORTANT: Press the plunger gently to avoid excess product extrusion.*
5. Insert gutta percha cone.
6. Radiograph to verify proper canal filling.
7. Cut the cone to desired height with heated instruments followed by vertical compaction.
8. Remove excess material from canal walls with appropriate irrigating solutions, perform coronal sealing and restoration.

B. INTERNAL RESORPTION TREATMENT

1. Anesthetize and place rubber dam.
2. Remove granulation tissue from resorption area using sharp curettes.
3. Neutralize the environment with calcium hydroxide paste or other intra-canal dressing as per manufacturer's instructions. Material should be used according to established use instructions by the manufacturer.

4. In the following session, remove calcium hydroxide or other intra-canal dressing.
5. Dry the canal only with absorbent paper points without causing excess dryness.
6. Insert BC SEALER ION⁺ with applicator tip throughout the canal, prioritizing the resorption site.
7. Introduce accessory gutta percha cones according to lateral compaction technique.
8. Cut cones at the top of the resorption area with heated instruments. Perform vertical compaction for better cement flow at resorption site.
9. Radiograph to verify proper filling of resorption site and canal.
10. Obtain remaining canal with gutta percha.
11. Perform coronal sealing with glass ionomer or other preferred material and restore tooth.
12. Radiograph and monitor for at least two years.

RETREATMENT

1. BC SEALER ION⁺ can be removed for retreatment or post-treatment.
2. Use preferred retreatment technique. Ultrasonic is indicated for more effective removal of obturation.

STORAGE

- Store the product in a dry, ventilated place between 15 and 30°C (60 and 85°F) with relative humidity below 60%.
- Do not refrigerate.

DISPOSAL METHOD

Dispose of the product in accordance with national laws and local authority recommendations for proper disposal.

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Brasseler U.S.A Dental, LLC