




BRASSLER
USA®

KontrolFlex™

Rotary Files

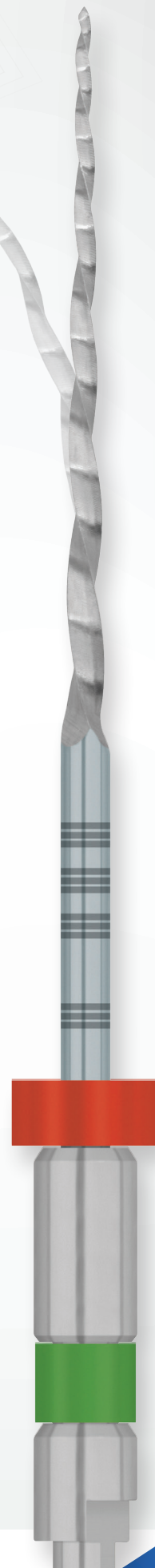

BY YOUR SIDE

TAKE CONTROL WITH KONTROLFLEX

Brasseler's KontrolFlex files provide superior control and safety in the most challenging canals. KontrolFlex files are available in two distinct metallurgies. KontrolFlex Series I (gold hub) is extremely ductile and ideal for difficult access and severe curvatures. KontrolFlex Series II (silver hub) can also be pre-curved but it is more robust and less likely to deform under torsional stress. Both versions will regain their original shape once heated (chairside with a heat source or autoclaved) as long as they have not been over stressed. It is recommended to use KontrolFlex files with a feather light touch cleaning the flutes frequently.



KONTROLFLEX™ Series I (4/PK)

- Highly Ductile
- Compare to HyFlex® CM™



KONTROLFLEX™ Series II (4/PK)

- More Robust
- Compare to Vortex Blue™ and ProTaper Gold™

TAKE CONTROL WITH BRASSELER'S KONTROLFLEX™ NITI FILES

- ▶ Featuring patented* controlled memory NiTi
- ▶ Excellent for extremely curved/calcified canals
- ▶ Can be pre-bent for difficult access or bypassing ledges
- ▶ Superior resistance to cyclic fatigue
- ▶ Compare to Vortex Blue™, ProTaper Gold™, V-Taper™ 2, HyFlex® CM™ and EdgeFile®.
- ▶ Now available in two distinct controlled memory metallurgies!

KONTROLFLEX™ CLINICAL TIPS


- ▶ Always reach full working length to at least a #15/.02 prior to use of rotary files.
- ▶ 500-600 RPMs.
- ▶ 1.8-2.3 Ncm.
- ▶ Use a feather-light touch taking the file to engagement (not resistance).
- ▶ Use the Single Stroke and Clean Technique. Clean the flutes after each gentle engagement.
- ▶ Irrigate between each rotary file.

OUR MOST FLEXIBLE FILE

KontrolFlex files can be utilized as a stand-alone shaping system or as adjunct files for specific clinical situations (difficult access, extreme curvatures, etc). The extensive range of tip sizes and tapers allows you to quickly incorporate the KontrolFlex files that best meet your treatment philosophy.


CONSTANT TAPER SHAPING

COMPARE TO: Vortex Blue™ and V-Taper™ 2

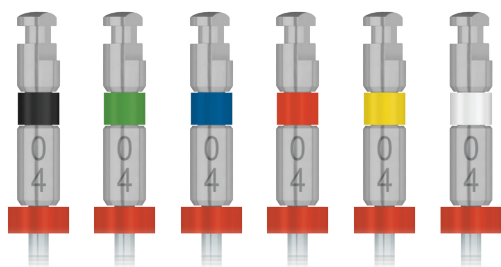

VARIABLE TAPER SHAPING

COMPARE TO: ProTaper Gold™

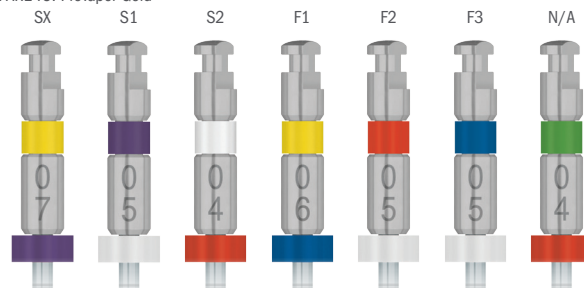

KONTROLFLEX™ PRODUCT OFFERING (4/PACK)

File/ Tip ID	Tip Size	Series I		Series II		Series III		Series IV							
		.04 Taper (Stopper)				.05 Taper (Stopper)		.06 Taper (Stopper)		.07 Taper (Stopper)					
		21 mm	25 mm	31 mm	21 mm	25 mm	21 mm	25 mm	19 mm						
△ White	15	Yes	No	No	No	No	No	No	No	No	No	No	No	No	No
▲ Purple	17	No	No	No	Yes	No	Yes	No	No	No	No	No	No	No	No
▲ Yellow	19	No	No	No	No	No	No	No	No	No	No	No	Yes	No	No
▲ Yellow	20	Yes	No	No	No	No	No	Yes	No	Yes	No	No	No	No	No
▲ Red	25	Yes	No	No	Yes	No	Yes	No	No	No	No	No	No	No	No
▲ Blue	30	Yes	No	No	Yes	No	Yes	No	No	No	No	No	No	No	No
▲ Green	35	Yes	No	No	No	No	No	No	No	No	No	No	No	No	No
▲ Black	40	Yes	No	No	No	No	No	No	No	No	No	No	No	No	No
△ White	45	Yes	No	No	No	No	No	No	No	No	No	No	No	No	No
Asst.	35-20	No	No	No	No	No	No	No	No	No	No	No	No	No	No


EdgeFile® is a registered trademark of Edge Endo, LLC.
V-Taper™ 2 is a trademark of Guidance Endodontics, LLC.
HyFlex® CM™ is a registered trademark of Coltene/Whaledent, Inc.
Vortex Blue™, and ProTaper Gold™ are registered trademarks of DENTSPLY International, Inc.



DIAMONDS



CARBIDES

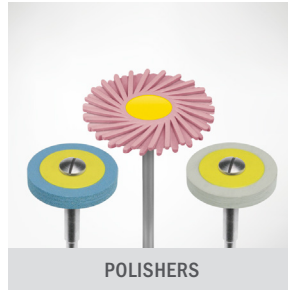

POLISHERS



PROCEDURE SYSTEMS



BIOCERAMICS



Since 1976, our absolute focus has been to develop products and provide services that support the practice of core operatory procedures. Today, Brasseler USA® is the premier dental instrumentation company in North America offering the most extensive selection of dental and surgical instrumentation under one brand.



HANDPIECES



ENDODONTICS



SMALL EQUIPMENT



HYGIENE



HAND INSTRUMENTS



ORAL SURGERY



LABORATORY INSTRUMENTS



Visit our website at BrasselerUSA.com

To order call 800.841.4522 or fax 888.610.1937. In Canada call 800.363.3838

