Indications:

EDTA solution (17%); necessary for the removal of the smear layer and dentin mud.

Instructions for use:

1. Select desired size syringe and irrigation tip.
2. Fill syringe with EDTA (10mL recommended).
3. Irrigate as desired.
4. Remove and dispose of irrigating tip from the syringe.
5. Dispose of syringe after use.

Use and Storage:

- For dental use only.
- This medical device should not be used after the expiration date.

Warnings:

- ** Not to be used as an oral rinse. **
- Always replace cap after use.
- Avoid contact with skin and eyes.
- Keep out of the reach of children.
- Store at room temperature.
- Not for injection.

WARNING

Causes mild skin irritation. Causes eye irritation. May be harmful if swallowed. Wash hands thoroughly after handling. Wear protective gloves and eye protection. Follow all government regulations for waste disposal. Prevent rework to the environment if possible. Do not flush waste into sewer or waterways. **FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do so. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water.

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Disodium
Ethylenediaminetetraacetate

17% EDTA Solution

For removal of smear layer
and dentin mud

Formulated with ideal pH for optimal chelation

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brasselerUSA.com ©2012