



BRASSELER
USA®

To order call 800.841.4522.

Visit our website at

BrasselerUSA.com

DualForce™

Sectional Matrix System

Matrix Forceps

REF 209024-B

QTY 1

(01)00887919891383

(10) XXXXX

LOT XXXXX

Rx Only

Made in US

Distributed by

Brasseler U.S.A. Dental, LLC

One Brasseler Blvd

Savannah, GA 31419, U.S.A.



BrasselerUSADental.com/Education-Resource/IFU/

REVO



powered by

**Clinician's
Choice®**

B-5498-I

DualForce™

Sectional Matrix System

INSTRUCTIONS FOR USE

- 1.** Before each use, wash, sterilize and inspect the DualForce Matrix Forcep.
- 2.** Acceptable sterilization methods include high pressure steam sterilization and dry heat sterilization.
- 3.** Please follow the instructions provided by the manufacturer of the sterilization equipment.

WARNING

Failure to follow the instructions may result in permanent damage to the forcep.

- 1.** It is the responsibility of the user to determine the suitability of the product for an intended procedure prior to use.
- 2.** Using the forcep outside of its intended use will result in damage or degradation.
- 3.** Inspect the forcep before each use. Discontinue use if found to be damaged.
- 4.** The user assumes all risk and liability in connection with damage or loss, either direct or consequential, stemming from the use or misuse of this product.