BRASSELER
USA®

••BRiO

LOWSPEED ACCESSORY

INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

Caution: Federal (US) law restricts this device to sale by or on the order of a licensed professional.

**Caution**

- Before each use, check for vibration, noise, and overheating. If any abnormalities are observed, stop using immediately and contact Brasseler USA.
- Do not drop, hit or subject to excessive shock.
- Sterilization is required before first use.
- Do not wipe or immerse in acidic solutions. The user is responsible for correct operation, maintenance, and inspection.
- Do not disassemble or alter the motor, attachments or heads.

SPECIFICATIONS**Heads**

ITEM CODE	DESCRIPTION	BURS
5028348U0	BRiO LS-KV-L	Latch-type (RA) Type 1
5028349U0	BRiO LS-KV-F	Friction grip (FG) Type 3
5028350U0	BRiO LS-ST-L	Latch-type (RA) Type 1

ITEM CODE	DESCRIPTION	BURS
5028351U0	BRI0 LS-ST-F	Friction grip (FG) Type 3
5028352U0	BRI0 LS-MW-L	Latch-type (RA) Type 1
5028353U0	BRI0 LS-MW-F	Friction grip (FG) Type 3

INSTALLATION

Attaching/Removing Heads From Contra Angle For BRI0 LS-ST and LS-MW Heads:

Attaching

1. Insert drive gear assembly into sheath. Be sure it seats properly.
2. Slide head union nut up to expose castellated fitting.
3. Lower head onto drive assembly and align head and sheath fittings. Make sure fittings interlock completely.
4. Securely tighten union nut and check for any looseness.

Removing

1. Loosen union nut and remove head from sheath.
2. Remove drive gear assembly.

For BRI0 LS-KV Heads:


Attaching

1. Turn and hold spring-loaded locking collar in the full open position.

2. Insert head fully and release locking collar.
3. Lightly pull on head to ensure that it is securely installed.

Removing

1. Turn and hold spring-loaded locking collar in the full open position.
2. Remove head and release locking collar.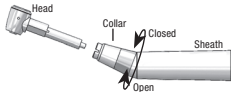

INSERTING/REMOVING BURS


Push-Button Latch Heads

Inserting bur

1. Press push button or slide latch upwards to open chuck.
2. Insert notched end of latch bur.
3. Slightly rotate bur until it engages with latch mechanism.
4. Release push button or latch to lock in bur.

Removing bur

1. Press push button or push latch upwards to open chuck.
2. Remove bur from head.
3. Release latch or push button.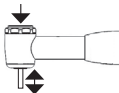


Push-Button Friction Grip Head

Inserting bur

1. Press push-button and insert bur into head until it stops.
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.

Removing bur

1. Press push-button on end cap with thumb.
2. Pull to remove bur.

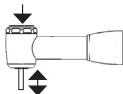STERILIZATION

- Sterilization is required before first use.
 - Sterilize by steam autoclave after each patient use.
 - Do not use ultrasonic bath, boiling water or chemicals to clean.
 - Do not autoclave at temperatures that exceed 275 °F (135 °C).
 - Do not autoclave together with other instruments.
 - Do not use dry heat sterilization.
 - Wear appropriate personal protective equipment during sterilization and maintenance process. This should include eye protection and gloves.
 - Keep in sterilization pouch until ready to use.
1. Wipe handpiece with damp cloth.
 2. Place handpiece in sterilization pouch and seal. Bag handpiece individually.
 3. Steam autoclave for a minimum of 15 minutes at 270 °F (132 °C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

SERVICE

For items that require service, including warranty service, please send to Brasseler USA.

Brasseler U.S.A. Dental, LLC
One Brasseler Blvd.,
Savannah, GA 31419, USA
Shop.brasselerusa.com
800.841.4522

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Brasseler USA will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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