BRASSELER
USA®

⋮BRiO**450**

⋮BRiO**450KL**

⋮BRiO**450L**

⋮BRiO**450F**

HIGH SPEED AIR TURBINE SURGICAL HANDPIECE
INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional

Brasseler U.S.A. Dental, LLC hereinafter will be referred to as “Brasseler USA.”

Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed
 Caution	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.

Symbols	
	Serial Number
	Reference Number
	Quantity
	Instructions for Use

INDICATION FOR USE

This air-powered dental handpiece is intended to be used for cavities and crown preparations, removal of fillings and prosthetic restorations, removal of carious tissue, and as a surgical tool for third molar removal procedures. It is designed for use by a trained professional in the field of general dentistry.

Caution

- Do not run handpiece without bur locked in chuck.
- Do not disassemble, modify or alter handpiece.
- Do not press push-button end cap during use since this could result in OVERHEATING of handpiece head. Special caution must be exercised during use to keep cheek tissue away from push-button end cap. Contact with cheek tissue may cause push-button to depress and may result in burn injury to patient.
- Never attempt to remove bur while handpiece is running.
- If any abnormalities are found, stop using immediately and contact Brasseler USA.

SPECIFICATIONS

Description	BRiO 450F	BRiO 450	BRiO 450L	BRiO 450KL
Item Code	5027992U0 / 5027992U2	5027990U0 / 5027990U2	5027989U0	5027991U0
Drive Air Pressure	29 – 33 psi	38 – 42 psi	38 – 42 psi	38 – 42 psi
Water Pressure	12-29 psi / 20 psi Recommended			
Spray Air Pressure	N/A			
Operating Speed	380,000 – 440,000 rpm	380,000 – 440,000 rpm	380,000 – 440,000 rpm	380,000 – 440,000 rpm
Connection Type	4-hole	Brasseler Non-Optic	Brasseler Optic	MULTIflex® Optic
Power	21 W	21 W	21 W	21 W
Weight	45 g	48 g	49 g	58 g
Head Height	12.35 mm	12.35 mm	12.35 mm	12.35 mm

Head Diameter	10.50 mm	10.50 mm	10.50 mm	10.50 mm
---------------	----------	----------	----------	----------

Checking Handpiece Pressure

1. Insert the test multi-gauge manometer between the handpiece/coupler and delivery system tubing. (Fig. 1).
2. Check the drive air pressure, water pressure, and the spray air pressure.

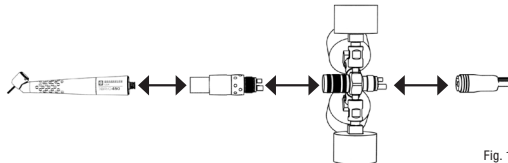
Fig. 1

Caution


- Exceeding recommended air pressure may result in premature cartridge failure.

OPERATION

BRIO 450 / BRiO 450L / BRiO 450KL

Connecting Coupler (Fig. 2)

1. Align tubes and pins on coupler with holes in delivery system tubing.
2. Thread tubing nut to bottom of coupler.
3. Hold coupler and finger tighten nut.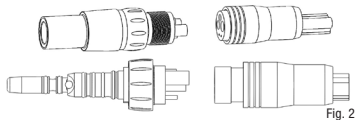

Connecting Handpiece (Fig. 3)


1. Slide handpiece onto coupler.
2. Press firmly until it snaps into place.

Note: Handpiece should swivel freely with slight tension and sit flush on coupler.

Removing Handpiece (Fig. 3)

Hold ring on coupler and pull handpiece straight off.

Note: Do not hold or pull on hose.

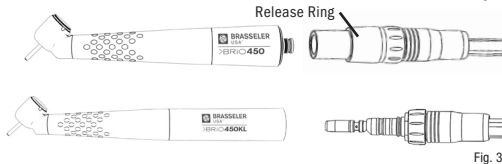BRiO 450F (Fig. 4)

Connecting Handpiece

1. Align tubes in handpiece with delivery system tubing.
2. Thread tubing nut to bottom of handpiece and finger tighten.

Removing Handpiece

1. Unthread tubing nut and pull handpiece straight off.

Fig. 4

Changing Burs (Fig. 5)

Caution

- Only use ISO standard burs with shank diameter of .0628" - .0630" (1.595mm-1.600mm) and an overall length of .9842" (25mm).
- Do not use standard length or short shank burs with these handpieces, they can come out and injure the patient or be swallowed.
- The water stream is designed to be delivered to the tip of a 25mm bur only. Using a bur of another size risks overheating the cutting area.
- Do not use bent, worn or damaged burs.
- Always keep burs clean. Dirt and debris on shank may cause clogging/sticking, loss of grip or premature wear of chuck.

Installation

1. Press push-button and insert bur into head until it stops.
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.

Removal

1. Press push-button on end cap with thumb.
2. Pull to remove bur.

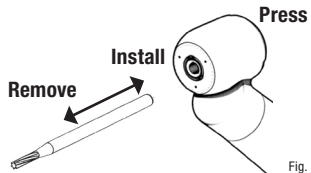
Fig. 5

MAINTENANCE

Caution

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.

Manual Cleaning and Lubrication

Handpiece Maintenance

Note: Cleaning and lubrication is required before each sterilization cycle.

1. Remove bur.
2. Detach handpiece from coupler or delivery system.
3. BRiO 450 / BRiO 450L / BRiO 450KL: Fit spray adapter onto universal cap nozzle of approved lubricant (Fig. 6).
4. Insert spray adapter (Fig. 7) or nozzle (Fig. 8) completely into back of handpiece and hold it in place.

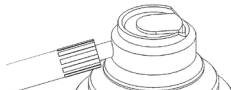
Fig. 6

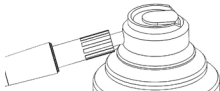
Fig. 7

5. Dispense spray for 1-2 seconds, then run handpiece to purge cleaner/lubricant.
6. Repeat steps 4 and 5 until cleaner/lubricant expelled from head is clear.
7. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

Chuck Maintenance

Note: It is recommended that the push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Detach handpiece from coupler or delivery system.
3. Fit chuck adapter onto universal cap nozzle of approved lubricant.
4. Insert spray adapter into chuck and dispense spray for 1 - 2 seconds (Fig. 9).
5. Work chuck mechanism to distribute cleaner/lubricant by repeatedly pressing button.
6. With push-button depressed, insert and remove bur several times to break-up remaining debris.
7. Wipe off excess cleaner/lubricant with a dry, lint-free cloth.

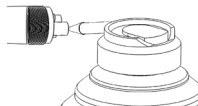
Fig. 8


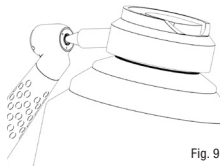
Fig. 9

Cleaning Water Spray Ports

Note: It is recommended that spray ports be periodically cleaned.

1. Carefully clean ports using cleaning wire supplied with handpiece (Fig. 10).
2. Connect handpiece and run with water flow activated to confirm ports have been fully cleared of debris.
3. Repeat steps 1 and 2 as necessary.

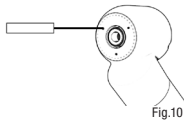Cleaning Optic Rod

1. Clear optic surfaces of particulates by spraying with air syringe.
2. Wipe optic surfaces with a soft cloth or cotton swab moist with alcohol.

Changing Coupler O-rings

- Improper care of o-rings may cause handpiece to malfunction.
- When changing/replacing coupler o-rings, always replace complete set.

Note: Coupler o-rings may be lubricated.

1. Use an explorer to gently remove old/worn o-rings.
2. Slide on new o-rings using only fingers to not cause damage.

STERILIZATION

WARNING

- Sterilize with autoclave after each patient use.
- Sterilization is required before first use.

Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
- Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
- Do not autoclave handpieces together with other instruments.
- Do not use dry heat sterilization.
- Keep handpiece in sterilization pouch until ready to use.
- Do not cool handpiece manually as damage may occur.
- Do not autoclave at temperatures that exceed 275° F (135° C).

1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in

bio-hazard bin when finished.

2. Place handpiece in sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.
3. Sterilize in a gravity type autoclave for a minimum of 15 minutes or in a pre-vacuum autoclave for a minimum of 4 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

REPLACEMENT PARTS & ACCESSORIES

5010059U0	High speed Spray Nozzle	5027293U0	BR-5 Coupler for 5 Hole
5010747U0	KaVo Spray Nozzle	5027001U0	BR-4 Coupler for 4 Hole
5027294U0	BR-6 LED Coupler for 6 Hole	5027003U0	BR-K6 LED KaVo Style Coupler for 6 Hole
5021281U0	NLT0TI Coupler for 6 Hole	5027002U0	BR-K5 KaVo Style Coupler for 5 Hole
5027157U0	BRiO Cap Wrench	5028249U0	BRiO 450 Turbine

SERVICE

For items that require service, including warranty service, please send to Brasseler USA.

Brasseler USA

Attn: Dental Repairs

One Brasseler Blvd.

Savannah, GA 31419

800.841.4522 (in Canada, call 800.363.3838)

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Brasseler USA will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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