BRASSELER
USA®

EVERclean
MAINTENANCE MACHINE


INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional.

ELECTRICAL SAFETY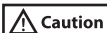

- Never operate unit if cord is damaged or loose.
- Keep unit away from water or other liquid. If the unit falls into water, unplug it immediately; do not reach into the water or liquid. Send to Brasseler USA for evaluation and repair.
- Do not use extension cord to operate.
- When unplugging unit, grasp plug with a dry hand. Never jerk cord.
- Do not use outdoors or in an extremely humid or damp environment.
- Never allow power cord to be pulled, twisted or severely bent.
- To avoid risk of electrical shock, equipment damage or fire during an electrical storm, turn machine off and do not touch machine or cord.
- This unit has a grounded plug as a safety feature. Never bypass.
- Keep unit away from excessive heat sources.
- Only use 115 V ac 60Hz input power.

GENERAL OPERATIONS/PRECAUTIONS

- For professional use only.
- Turn off unit before unplugging.
- Wear appropriate personal protective equipment during use and cleaning.
- Use only with adequate ventilation.
- Sterilize handpieces after each maintenance cycle.
- Keep workspace clean and free of debris.
- Only use accessories approved for use with this product.
- If any abnormalities are found, stop using immediately and contact Brasseler USA.
- Do not expose to direct heat or open flame.

INDICATIONS FOR USE

The EVERclean Maintenance Machine is intended for cleaning and lubrication prior to sterilization of high-speed, low-speed, and electric handpieces as well as cartridge-style sonic air scalars.

SYMBOLS


Not General Waste


Shock Protection
Type B

PARTS AND CONTROLS

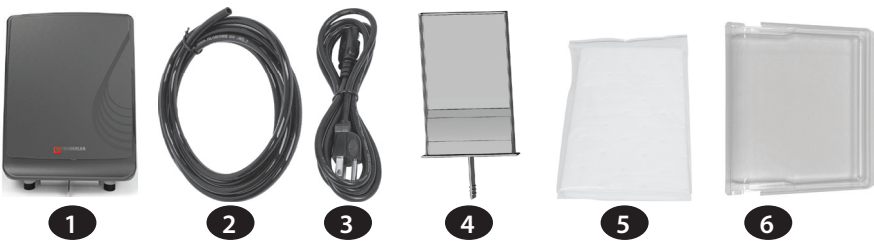In the Box 1. Main Unit 2. Air Tube 3. Power Cord 4. Oil Tray 5. 3-Pack Oil Sheets 6. Door Shield	
Main Unit - External 7. Control Panel 8. Front Door 9. Lubricant Door 10. Power Switch 11. Fuse Holder 12. Power Cord Connection 13. Air Connection 14. Air Regulator	
Main Unit - Internal 15. 4-hole Connectors 16. Chuck Nozzle 17. Internal Oil Suction	
Control Panel 18. Handpiece Selector Lights 19. Handpiece Selector Buttons 20. LCD Display 21. Start Button 22. Empty Spray Can Warning Light	

ASSEMBLY SETUP

Note: Leave adequate space around the EVERclean for all connections. The EVERclean should be positioned on a firm, flat, vibration-free surface.

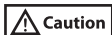- Air supply must be clean and oil free.
- Make sure the valve for main air supply is not opened until set up is complete.
- Oil may leak if spray can is not properly seated. If can is loose, adjust can holder by unscrewing or turning to the left.
- Use only EVERclean spray. Using other cleaners may damage unit.
- Make sure 4-hole connector nut is securely tightened when handpiece or service adapter is attached to avoid creating an air and/or oil leak.

1. Insert air tube completely into air connection on back of unit.
2. Connect other end of air tube to main air supply.

Note: Air Regulator dial is pre-set to 4.5 bars. Recommended air pressure is 4.5 bars. Air pressure outside 4 - 6 bars will result in an error code.

3. Connect one end of power cord to the power cord connection and plug other end into grounded electrical outlet.
4. If using service adapters, insert adapter into 4-hole connector & tighten nut (Fig. 1).

FIG. 1

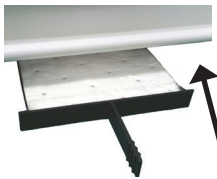
FIG. 2

FIG. 3

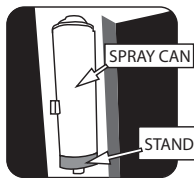
FIG. 4

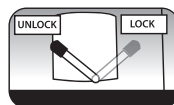
FIG. 5

5. Place 1 oil sheet in oil tray and insert completely into lower front of unit (Fig. 2).
6. Press the "push" label on lubricant door to open (Fig. 3).
7. Move lever into unlocked position (Fig. 5), then place EVERclean Spray (5028210U0) on stand (Fig. 4).
8. Ensure can nozzle is properly seated, then move lever into locked position (Fig. 5).
9. Open main air valve to pressurize maintenance unit.
10. Turn power switch on.

OPERATIONAL SETUP

1. Press Start button for 2 seconds to enter set-up mode.
2. "Compressed air on" will appear on LCD display confirming compressed air is on.
3. Press Start button to show handpiece purge/dry time. The default time is set to 45 seconds. Press "1" to decrease or "3" to increase purge/dry time as needed.
NOTE: There are six time settings. (30s/ 45s/ 60s/ 75s/ 90s/ 120s).
4. Press Start button again to display the buzzer setting. If the buzzer is set to "off," press "1" to turn buzzer on.
5. Press Start button again to display compressed air setting. The compressed air setting should show 4.5 bar. If LCD display does not show 4.5 bar, turn the Air Regulator counterclockwise to lower or clockwise to increase the air pressure until it is set at 4.5 bars.

OPERATION

- Wear appropriate personal protective equipment during use and cleaning.

Chuck maintenance

Note: Ensure bur is removed from handpiece before attempting this step. It is recommended that handpiece or attachment chuck be cleaned every 3 - 4 maintenance cycles.

1. Open front door.
2. Place chuck opening against tapered end of chuck cleaning nozzle (Fig. 6).
3. Press Start. LCD display will illuminate while in use.
4. Keep handpiece in place until beep sounds indicating that cleaning cycle is completed.

FIG. 6

Handpiece and attachment maintenance

Attaching high-speed air-driven handpieces

- Ensure bur is removed from air-driven high-speed handpieces before using maintenance machine.
- Missing or improperly seated o-rings on service adapter may cause handpiece to become stuck on adapter or o-ring(s) may become damaged.

1. Open front door of maintenance machine.
For quick connect style handpieces only
2. Attach required service adapter by inserting back of adapter into 4-hole connector and finger tighten threaded nut.
3. Inspect service adapter to ensure that all o-rings are properly seated in their grooves.
4. Push handpiece onto applicable service adapter.
For fixed backend handpieces only
5. Insert fixed backend handpiece into 4-hole connector and finger tighten nut.

Attaching straight and contra angle attachments

1. Open front door.
2. Attach required low-speed service adapter (if applicable) by inserting back of adapter into 4-hole connector and finger tighten threaded nut.
3. Push attachment onto applicable service adapter until it clicks and locks in place.

Handpiece and attachment lubrication and internal cleaning

- Be sure to only select maintenance station(s) that have handpiece(s) installed.

Note: Do not depress buttons or open front door of EVERclean while machine is running, this will cause maintenance cycle to be interrupted. If this occurs, open door completely to reset machine and then start lubrication and internal cleaning cycle again.

1. Close front door of maintenance machine.
2. Select maintenance station number(s) on the control panel by pressing the 1, 2 and/or 3 button(s) that correspond to stations with handpiece(s) attached. Indicator lights will illuminate when station is activated for use.
3. Press Start button. Machine is now operating. LCD display will illuminate while in use.
4. Machine will beep when maintenance cycle is complete.
5. Remove handpiece(s) and/or attachment(s) from maintenance machine. To disconnect attachment(s), press the lever on the low-speed service adapter before removing.
6. Place handpiece(s) in sterilization pouch(es) and sterilize according to manufacturer recommendation.

MACHINE CLEANING AND MAINTENANCE

- Do not use hot water, solvents, chemicals or auto-sterilizer for cleaning machine or accessories.
- Wear appropriate personal protective equipment while cleaning unit.
- Only use EVERclean Oil Sheets (5028209U0) in the oil tray. The use of any other substance may result in oil leakage or gassing of the machine.

Maintenance Unit

Note: It is recommended that the maintenance unit be wiped down every evening.

1. Turn off power to maintenance machine and unplug unit from wall.
2. Wipe down all internal and external surfaces with a soft damp cloth or soft damp paper towel.
3. Discard cloth or paper towel in biohazard bin.

Door Shield

Note: It is recommended that the door shield (5028220U0) be replaced when it starts to yellow or change color.


1. Open front door and remove protective door shield.
2. Clean door shield by rinsing with lukewarm tap water or isopropyl alcohol and wipe dry with paper towel. Discard paper towel in biohazard bin.
3. Install clean protective door shield.

Oil Tray

Note: It is recommended that oil sheets (5028209U0) be replaced every 2 weeks.

1. Remove oil tray from bottom of machine.
2. Remove old oil sheet and discard in biohazard bin.
3. Clean tray by rinsing with lukewarm tap water or isopropyl alcohol and wipe dry with paper towel. Discard paper towel in biohazard bin.
4. Place new oil sheet into tray and insert oil tray into machine.

Service Adapters

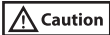- Missing or improperly seated o-rings on service adapter may cause handpiece to become stuck on adapter or o-ring(s) may become damaged.

Note: It is recommended that service adapters be cleaned weekly.

1. Remove service adapter from threaded connector.
2. Clean service adapter by rinsing with lukewarm tap water or isopropyl alcohol and wipe dry with paper towel. Discard paper towel in biohazard bin.
3. Inspect service adapter to ensure that all o-rings are properly seated in their grooves.
4. Attach service adapter following instructions in Step 4 in ASSEMBLY SET UP.

EVERclean Spray

FIG. 7

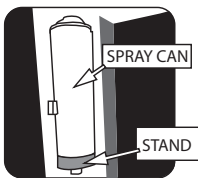
FIG. 8

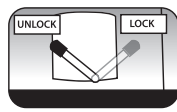
FIG. 9

1. Turn off power to maintenance machine and unplug unit from wall.
2. Press the "push" label on lubricant door to open (Fig. 7).
3. Move lever into unlocked position (Fig. 9) while holding spray can to help guide it down.
4. Remove empty spray can and dispose according to local regulations.
5. Place EVERclean Spray (5028210U0) on stand (Fig. 8).
6. Ensure can nozzle is properly seated, then move lever into locked position (Fig. 9).

Fuses

1. Turn off power to maintenance machine and unplug unit from wall.
2. Insert small flathead screwdriver into divot on side of fuse holder to press tab inward and lift out fuse holder (Fig 10).
3. Pull out fuse(s).
4. Install new fuse(s) (250 V / 2 A slow blow) into holder.
5. Place fuse holder back into unit.

FIG. 10

ACCESSORIES

5028210U0 EVERclean Spray Can

5028209U0 10-pack Oil Sheets

5028220U0 Door Shield

Service Adapters

5028211U0 KaVo

5028217U0 Midwest High-speed

5028219U0 Midwest Low-speed

5028212U0 A-dec / W&H

5028218U0 Star

5028213U0 Brasseler / NSK

5028214U0 E-type

5028216U0 Sirona

TROUBLESHOOTING

Note: The EVERclean Maintenance Machine will beep continuously any time there is an operation error. Before attempting any troubleshooting solutions, power off machine.

PROBLEM	CAUSE	SOLUTION
Unit will not operate; lights and LCD display not on.	Unit not powered on.	Check unit power cord connections at outlet and back of unit.
		Make sure power outlet is live.
		Inspect and change fuse(s).
Unit will not operate when Start button pressed; LCD display shows "DOOR."	Front door open.	Close front door.
Unit will not operate when Start button pressed; LCD display shows "CHECK CAN" and Empty Spray Can light is blinking.	EVERclean Spray can empty.	Replace EVERclean Spray can.
	EVERclean Spray can not properly installed.	Reinstall EVERclean Spray can.
Unit will not operate when Start button pressed; LCD display shows "Low air pressure. Code 04."	Compressed air is not connected.	Properly connect compressed air.
	Air pressure set below 4 bars.	Enter OPERATIONAL SETUP for air pressure and use air regulator to adjust to 4 - 6 bars.
Unusual odor	Oil tray liners saturated.	Replace oil tray liners and clean oil tray.
Spray can will not install.	Spray nozzle is bent.	Replace spray can.
	Not using EVERclean Spray.	Use only EVERclean Spray.

ERROR CODE & TYPE	DESCRIPTION	SOLUTION
Error 01 Keyboard Error	Control panel malfunction	Contact Brasseler USA
Error 02 Oil supply motor error	Oil inlet valve malfunction	Contact Brasseler USA
Error 04 Pressure input too low	Air pressure too low	Check that air supply pressure is set to at least 4 bars as explained in the OPERATIONAL SETUP section
Error 05 Pressure input too high	Air pressure too high	Check that air supply pressure is set to less than 6 bars as explained in the OPERATIONAL SETUP section
Error 06 Function memory access error	Electromagnetic switch malfunction	Contact Brasseler USA
Error 07 Oil empty	EVERclean Spray can empty	Replace spray can
Error 08 Electromagnetic valve error	Oil not being distributed to handpieces/ attachments	Contact Brasseler USA
Error 09 Circuit is over temperature	Circuit board malfunction	Contact Brasseler USA

ERROR CODE & TYPE	DESCRIPTION	SOLUTION
Error 10 Circuit detect error	Circuit board malfunction	Contact Brasseler USA

SPECIFICATIONS

Power:	AC 100-240 V 50/60 Hz
Fuse:	250V 2A Slow Blow Type
Air Pressure:	58 -87 psi max (4 -6 bar)
Recommended Pressure:	65.3 psi (4.5 bar)
Air Consumption:	40 L _n /min
Weight:	19.8 lbs (9 kg)
Dimensions:	295 (W) x 400 (H) x 295 (D)
IP Code:	IP20
Classification (EN60601):	Class B
Operating Temperature:	59 ~ 104°F (15° ~ 40°C)
Transport & Storage Temperature:	-4 ~ 158F (-20° ~ 70°C)
Operating Humidity:	25 to 90%
Transport & Storage Humidity:	5 to 95%
Operating Altitude:	Below 6500 ft (2000 m)
Transport & Storage Pressure:	700 to 1060 hPa

DISPOSAL OF MEDICAL DEVICES

Any medical devices which could possibly be contaminated must first be decontaminated by the responsible doctor or medical institution and then be disposed of by an agent licensed and qualified to handle medical and industrial waste.

INFORMATION ON ELECTROMAGNETIC COMPATIBILITY

Guidance and manufacturer's declaration-electromagnetic emissions		
The EVERclean is intended for use in the electromagnetic environment specified below. The customer or the user of the EVERclean should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The EVERclean uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The EVERclean is suitable for use in all establishments, including domestic establishments, and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Compliance	

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 KHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the EVERclean including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80MHz to 2,5 GHz	3 V/m	

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the EVERclean is used exceeds the applicable RF compliance level above, the EVERclean should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the EVERclean.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

SERVICE

For items that require service, including warranty service, please send to Brasseler USA.

Brasseler USA
 Attn: Dental Repairs
 One Brasseler Blvd.
 Savannah, GA 31419
 800.841.4522 (in Canada, call 800.363.3838)

WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for 2 years from date of purchase.

Brasseler USA will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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