BRASSELER
USA®

REF 5027697U0

enLITE

INSTRUCTIONS FOR USE
INSTRUCCIONES DE USO
DIRECTIVES D'UTILISATION

Read before use. Keep in a safe place for future reference.

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CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional

Caution

- For professional use only.
- Light guide can become hot during use, avoid contact with it.
- Never look directly into the LED light source without eye protection.
- Sterilize light guide prior to first use.
- To avoid damage to the LED handpiece, do not drop or subject it to excessive shock.
- Keep away from water and other liquids.
- If charger base falls into water, unplug immediately. Do not reach into water or liquid.
- If either the charger or the LED handpiece come into contact with water or other liquids, stop using immediately and send to Brasseler USA for evaluation.
- Do not disassemble handpiece.
- Keep away from heat and direct sunlight.
- If any abnormalities are found, stop using immediately and contact Brasseler USA.

SPECIFICATIONS

CHARGER BASE

Input power: 100 ~ 240 V ac 50/60 Hz

Output power: 9 V 1.3 A 11 W

Weight: 200 g

LED HANDPIECE

Light intensity: 1600 mW/cm²


Wavelength: 450 ~ 470 nm (Dominant Range)

Weight: 160 g

PARTS AND CONTROLS

LED Handpiece

- 1. Light Guide 8 mm
- 2. Start/Stop Button
- 3. Set Time Button
- 4. DC Charger Input
- 5. Nose Cone
- 6. Housing

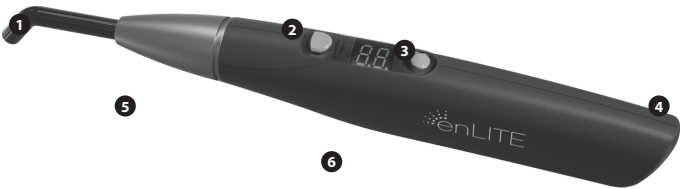
Charger

- 1. DC Charging Base
- 2. Charge Indicator Light
- 3. DC Adapter


Accessories

- 1. Glare Shield
- 2. Nose Cone O-ring
- 3. Anti-glare Cups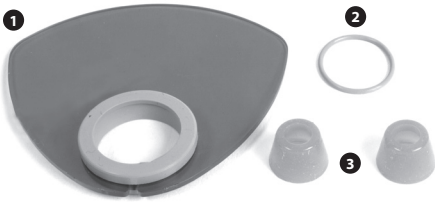

SETUP

- 1. Insert the light guide into the handpiece until it clicks securely into place. About 2 mm of metal on the light guide will remain visible.
- 2. Place the orange anti-glare cup onto the light guide and adjust according to preference.
- 3. Connect the DC adaptor to the charger stand and then plug into outlet.
Note: When first plugged in, the charging light will flash RED, then turn to a GREEN steady light which indicates the base is ready to charge.
- 4. Install glare shield onto the nose cone with the thicker side of the gasket towards the handpiece. Push the shield towards the handpiece until the flange inside the gasket is seated in the groove at the back of the nose cone.
Note: If the glare shield will not be used, install the supplied o-ring in the groove at the back of the nose cone to prevent debris build-up.
- 5. Place the handpiece in the charger, as shown, to begin charging. While in the base, the handpiece display will flash an alternating dash (" - ") pattern.

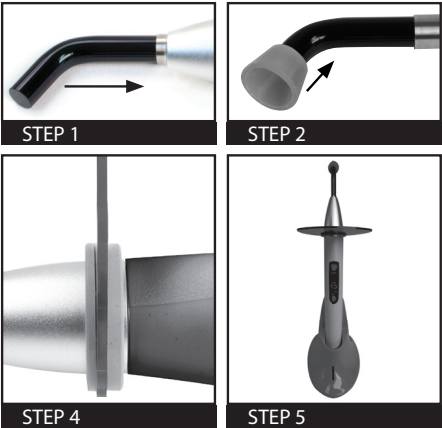Note: Allow the handpiece to fully charge. The battery should be charged for 5 hours prior to first use.

OPERATION

 **Caution**

- Wear protective eyewear and ensure glare shield is secured in place before use.
- Light guide can become hot during use, avoid contact with it.

On / Off

1. Press the “Start/Stop” button (A) once to turn on the display panel. The digital display will show the curing time from the previous cycle.
2. Press the “Start/Stop” button to begin or stop the curing cycle.
3. Two (2) beeps will countdown the last 2 seconds of the cycle and a 3rd beep will sound at the end of the cycle.

Note: The device will automatically switch off after being left at rest for 30 seconds. Press the “Start/Stop” button to turn the power back on.

Setting curing time

1. To program mode, press and hold the “Mode” button (B) until the display flashes.
2. Push the “Mode” button to increase the time in 5-second increments. When the time reaches 15 seconds, pressing the “Mode” button again results in the display rolling over to the minimum setting of 5 seconds.

Press	Time Setting	Mode
1	5 Seconds	Full Power
2	10 Seconds	Ramp Up
3	15 Seconds	Pulse

3. Press the “Start/Stop” button (A) to enter the new curing mode.

Screen displays

HH (Overheating): The handpiece is equipped with a sensor that will automatically turn off the light to prevent overheating. This may occur during long periods of high use. The handpiece will remain off for 6 minutes to allow for complete cooling.

Note: The EnLITE is not suitable for whitening applications.

bL (Low Battery): When the battery power is low, “bL” will appear in the display and the handpiece will start beeping. Place the handpiece in the charger to recharge battery.

Charging

 **Caution**

- Ensure handpiece is completely dry before placing in charger.

Charger ready: Steady green light (without handpiece in base)

Charging in progress: Steady red light. Handpiece will display an alternating dash “-” pattern

Charging complete: Steady green light (while handpiece in base)

Note: Completely charge battery at least every 2 months to maintain optimum battery life and performance. If battery needs to be replaced, contact Brasseler USA.

CLEANING AND STERILIZATION

 **Caution**

- Sterilize light guide before first use.
- Sterilize light guide after each patient use.
- Keep in sterilization pouch until ready to use.
- Wear appropriate personal protective equipment during the cleaning and sterilization process. This should include eye protection and gloves.
- Do not run or submerge handpiece in water.
- Do not sterilize handpiece body.

1. Wipe handpiece surface with a soft cloth soaked in warm water or a mild detergent mixture.
2. Remove light guide, place in sterilization pouch and seal. Bag light guides separately.
3. Dry handpiece completely before returning it to the charger.

TROUBLESHOOTING

PROBLEM	CAUSE	SOLUTION
Display is blank after pressing the “Start/Stop” button	Handpiece is not charged	Charge handpiece
	Defective battery	Contact Brasseler USA
	Integrated circuit board is defective	Contact Brasseler USA
Handpiece doesn’t charge	DC adapter is not properly connected	Check outlet and charger base connections
	Handpiece is inserted backward in charger	Reposition handpiece as shown in the setup section
	Poor connection between handpiece and charger	Check handpiece and base charging contacts to make sure they are clean and debris free
	Defective battery	Contact Brasseler USA
	Defective charging base	Contact Brasseler USA

SERVICE

For items that require service, including warranty service, please send to Brasseler USA.

Brasseler USA
1 Brasseler Blvd.
Savannah, GA 31419
Phone: 1-800-841-4522

WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for one (1) year from date of purchase.

Brasseler USA will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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